

Allē

Join Allē today for exclusive savings on SKINVIVE™ by JUVÉDERM®. Learn more at ALLE.COM

#### SKINVIVE™ by JUVÉDERM® Injectable Gel Important Information

##### APPROVED USES

SKINVIVE™ by JUVÉDERM® injectable gel is an injection to improve skin smoothness of the cheeks in adults over the age of 21.

##### IMPORTANT SAFETY INFORMATION

###### Are there any reasons why I should not receive SKINVIVE™ by JUVÉDERM® treatment?

Do not use this product if you have a history of multiple severe allergies or severe allergic reactions (anaphylaxis), if you are allergic to lidocaine or the Gram-positive bacterial proteins used in this product, or if you have had previous allergic reactions to hyaluronic acid fillers.

###### What Warnings should my specialist advise me about?

- One of the risks with dermal filler injection is the unintentional injection into a blood vessel. The chances of this happening are very small, but if it does happen, the complications can be serious and may be permanent. These complications, which have been reported for facial injections, can include vision abnormalities, blindness, stroke, temporary scabs, or permanent scarring of the skin. Most of these events are irreversible. Tell your specialist immediately if you have changes in your vision, signs of a stroke (including sudden difficulty speaking, numbness or weakness in your face, arms or legs, difficulty walking, face drooping, severe headache, dizziness, or confusion), white appearance of the skin, or unusual pain during or shortly after treatment
- The use of this product where skin sores, pimples, rashes, hives, cysts, or infections are present should be postponed, as this may delay healing or make skin problems worse
- The effectiveness of removal of any dermal filler has not been studied

###### What Precautions should my specialist advise me about?

- Minimize strenuous exercise, exposure to extensive sun or heat, and alcoholic beverages within the first 24 hours following treatment. Exposure to any of these may cause temporary redness, swelling, and/or itching at the injection site
- Tell your specialist if you are using any medication that can prolong bleeding, such as aspirin, ibuprofen, or other blood thinners, as this may increase bruising or bleeding at the injection site
- Tell your specialist if you are planning laser treatment, chemical peeling, or any other procedure after SKINVIVE™ by JUVÉDERM®. There is a possible risk of an inflammatory reaction at the treatment site
- This product is intended for improving skin smoothness of the cheeks. The safety and effectiveness for treatment in other areas of the body have not been established
- Tell your specialist if you are on therapy used to decrease the body's immune response, as treatment may result in an increased risk of infection
- Tell your specialist if you are pregnant or breastfeeding. The safety for use during pregnancy, or in women who are breastfeeding, has not been studied
- Tell your specialist if you have a history of excessive scarring (thick, hard scars). The safety of this product in patients with a history of excessive scarring has not been studied and may result in additional scars
- Tell your specialist if you have a history of pigmentation disorders, as use of this product in patients with a history of pigmentation disorders has not been studied and may result in changes in pigmentation

###### What are the possible side effects of treatment?

The most commonly reported side effects were redness, lumps/bumps, swelling, bruising, pain, tenderness, firmness, discoloration and itching. Most side effects will resolve within 7 days. If they persist longer, your physician may choose to treat them with medications, such as antibiotics, steroids, or hyaluronidase. Additionally, there have been reports of inflammation, nodules, unsatisfactory result, loss or lack of improvement, allergic reaction, anxiety, blood vessel blockage, infection, dry skin, increase or decrease in sensation, and abscess.

Delayed-onset inflammation near the site of dermal filler injections is one of the known adverse events associated with dermal fillers. As with all skin injection procedures, there is a risk of infection.

**To report a side effect, please call the Allergan® Product Support Department at 1-877-345-5372. Please also visit SKINVIVE.com or talk to your specialist for more information.**

SKINVIVE™ by JUVÉDERM® is available only by a licensed physician or properly licensed practitioner.

Allergan  
Aesthetics  
an AbbVie company

© 2023 AbbVie. All rights reserved. JUVÉDERM, SKINVIVE, and their designs are trademarks of Allergan Holdings France SAS, an AbbVie company, or its affiliates. All other trademarks are the property of their respective owners. SKV155384 07/23

 SKINVIVE™ BY JUVÉDERM®



Lasting  
Glow\*

**THE FIRST AND ONLY**  
hyaluronic acid microdroplet injectable indicated  
to improve skin smoothness of the cheeks.

Actual patient. Results may vary.

\*Lasts 6 months with optimal treatment. Glow was reported by patients using a validated questionnaire that included satisfaction with how radiant their skin looked.

##### APPROVED USES

SKINVIVE™ by JUVÉDERM® injectable gel is an injection to improve skin smoothness of the cheeks in adults over the age of 21.

**Please see Important Safety Information on the back cover.**

 SKINVIVE™ BY JUVÉDERM®



### Lasting 6-month Glow\*

SKINVIVE™ by JUVÉDERM® is a smooth hyaluronic acid (HA) gel that's injected as microdroplets below the surface of the skin, increasing hydration to improve cheek smoothness. In clinical studies, patients who were treated with SKINVIVE™ by JUVÉDERM® reported high satisfaction with how glowing, hydrated, refreshed, and healthy their skin looked through 6 months.



### Hydrated skin is essential to healthy skin

As you age, your skin loses HA and overall smoothness. HA hydrates the skin, improving its smoothness so the skin glows. SKINVIVE™ by JUVÉDERM® is a modified form of HA that helps the skin retain its natural moisture which improves skin smoothness and fine lines of cheeks.



### One treatment, 15 minutes†

One SKINVIVE™ by JUVÉDERM® treatment session typically lasts about 10–15 minutes, on average, and results last up to 6 months.\* It smooths and hydrates to help improve skin quality—approved for use in all skin types. Plus, treatment typically doesn't disrupt everyday activities.

Baseline

4 Weeks



*No makeup or retouching was used in these photos and lighting levels were consistent. 1 mL of SKINVIVE™ by JUVÉDERM® was injected into each cheek.*

**Ask your skin specialist about SKINVIVE™ by JUVÉDERM® today.**

\*Lasts 6 months with optimal treatment. Glow was reported by patients using a validated questionnaire that included satisfaction with how radiant their skin looked.

†A touch up treatment may be required to achieve desired goals.

**Please see Important Safety Information on the back cover.**

“After the treatment,  
my skin is just glowing.”

—Jennifer, a real patient

Actual patients. Results may vary.

# SKINVIVE Consumer Brochure

SKV155384

Finished size 4" x 8"

Flat size 8" x 8"

## Front Cover

4"

**SKINVIVE** BY JUVEDERM

**Lasting Glow**

**THE FIRST AND ONLY**  
hyaluronic acid microdroplet injectable indicated to improve skin smoothness of the cheeks.

Actual patient. Results may vary.

\*Lasts 6 months with optimal treatment. Glow was reported by patients using a validated questionnaire that included satisfaction with how radiant their skin looked.

**APPROVED USES**  
SKINVIVE™ by JUVEDERM™ injectable gel is an injection to improve skin smoothness of the cheeks in adults over the age of 21.

**Please see Important Safety Information on the back cover.**

8"

## Inside Spread

8"

**SKINVIVE** BY JUVEDERM

**Lasting 6-month Glow\***  
SKINVIVE™ by JUVEDERM™ is a smooth hyaluronic acid (HA) gel that's injected as microdroplets below the surface of the skin, increasing hydration to improve cheek smoothness. In clinical studies, patients who reported high satisfaction with how glowing, hydrated, refreshed, and healthy their skin looked through 6 months.

**Hydrated skin is essential to healthy skin**  
As you age, your skin loses HA and overall smoothness. HA hydrates the skin, improving its smoothness so the skin glows. SKINVIVE™ by JUVEDERM™ is a modified form of HA which improves skin smoothness and fine lines of cheeks.

**One treatment, 15 minutes†**  
One SKINVIVE™ by JUVEDERM™ treatment on average, and results last up to 6 months\*. It smooths and hydrates to help improve skin quality—approved for use in all skin types. Plus, treatment typically doesn't disrupt everyday activities.

**Baseline**      **4 Weeks**

*“After the treatment, my skin is just glowing.”*  
—Jennifer, a real patient

Actual patient. Results may vary.

No evidence of microdroplet gel used in these photos and glowing looks were observed 4 weeks post-treatment.

**Ask your skin specialist about SKINVIVE™ by JUVEDERM™ today.**  
\*Lasts 6 months with optimal treatment. Glow was reported by patients using a validated questionnaire that included satisfaction with how radiant their skin looked. †Touch up treatment may be required to achieve desired glow.

**Please see Important Safety Information on the back cover.**

8"

## Back Cover

4"

**Allé**  
Join Allé today for exclusive savings on SKINVIVE™ by JUVEDERM™. Learn more at [ALLE.COM](http://ALLE.COM)

**SKINVIVE™ by JUVEDERM™ injectable Gel Important Information**

**APPROVED USES**  
SKINVIVE™ by JUVEDERM™ injectable gel is an injection to improve skin smoothness of the cheeks in adults over the age of 21.

**IMPORTANT SAFETY INFORMATION**

**Are there any reasons why I should not receive SKINVIVE™ by JUVEDERM™ treatment?**  
Do not use this product if you have a history of multiple severe allergies or severe allergic reactions (anaphylaxis). If you are allergic to folic acid or the Gram-positive bacterial proteins used in this product, or if you have had previous allergic reactions to hyaluronic acid fillers.

**What Warnings should my specialist advise me about?**

- One of the risks with dermal filler injections is the unintentional injection into a blood vessel. The chances of this happening are very small, but if it does happen, the complications can be serious and may be permanent. These complications, which have been reported for facial injections, can include vision abnormalities, blindness, stroke, temporary scabs, or permanent scarring of the skin. Most of these events are irreversible. Tell your specialist immediately if you have changes in your vision, signs of a stroke (including sudden difficulty speaking, numbness or weakness in your face, arms or legs, difficulty walking, face drooping, severe headache, dizziness, or confusion), white appearance of the skin, or unusual pain during or shortly after treatment.
- The use of this product where skin sores, pimples, rashes, hives, cysts, or infections are present should be postponed, as this may delay healing or make skin problems worse.
- The effectiveness of removal of any dermal filler has not been studied.

**What Precautions should my specialist advise me about?**

- Minimize strenuous exercise, exposure to excessive sun or heat, and alcoholic beverages within the first 24 hours following treatment. Exposure to any of these may cause temporary redness, swelling, and/or itching at the injection site.
- Tell your specialist if you are using any medication that can prolong bleeding, such as aspirin, ibuprofen, or other blood thinners, as this may increase bruising or bleeding at the injection site.
- Tell your specialist if you are planning laser treatment, chemical peeling, or any other procedure after SKINVIVE™ by JUVEDERM™. There is a possible risk of an inflammatory reaction at the treatment site.
- This product is intended for improving skin smoothness of the cheeks. The safety and effectiveness for treatment in other areas of the body have not been established.
- Tell your specialist if you are on therapy used to decrease the body's immune response, as treatment may result in an increased risk of infection.
- Tell your specialist if you are pregnant or breastfeeding. The safety for use during pregnancy, or in women who are breastfeeding, has not been studied.
- Tell your specialist if you have a history of excessive scarring (thick, hard scars). The safety of this product in patients with a history of excessive scarring has not been studied and may result in additional scars.
- Tell your specialist if you have a history of pigmentation disorders, as use of this product in patients with a history of pigmentation disorders has not been studied and may result in changes in pigmentation.

**What are the possible side effects of treatment?**  
The most commonly reported side effects were redness, temporary swelling, bruising, pain, tenderness, firmness, discoloration and itching. Most side effects will resolve within 7 days. If they persist longer, your physician may choose to treat them with medications, such as antibiotics, steroids, or hyaluronidase. Additionally, there have been reports of inflammation, nodules, unsatisfactory result, loss or lack of improvement, allergic reaction, unsightly blood vessel blockage, infection, dry skin, increase or decrease in sensation, and abscess.

Delayed-onset inflammation near the site of dermal filler injections is one of the known adverse events associated with dermal fillers. As with all skin injection procedures, there is a risk of infection.

**To report a side effect, please call the Allergan® Product Support Department at 1-877-345-5372. Please also visit SKINVIVE.com or talk to your specialist for more information.**

SKINVIVE™ by JUVEDERM™ is available only by a licensed physician or properly licensed practitioner.

© 2023 Allergan. All rights reserved. JUVEDERM, SKINVIVE, and their designs are trademarks of Allergan Holdings France SAS, an Allergan company, or its affiliates. All other trademarks are the property of their respective owners. SKV155384 07/23

Allergan  
Aesthetic Division  
of Allergan Inc.